


# myobrace®

MYOFUNCTIONAL ORTHODONTICS


## ADULT ALIGNER

Arch development, dental alignment and habit correction in the permanent dentition

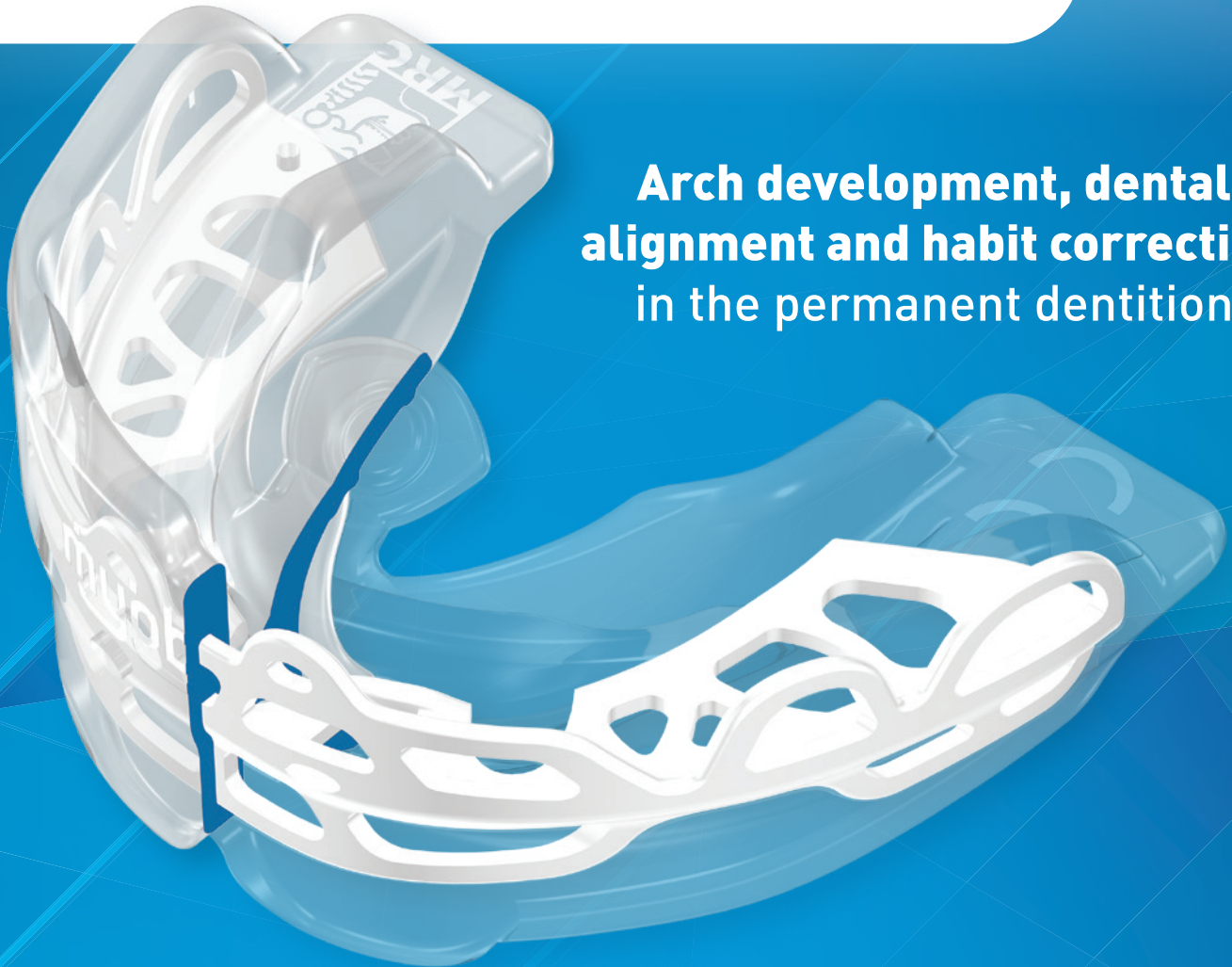

AA<sup>1</sup>  
ALIGNER

STAGE ONE

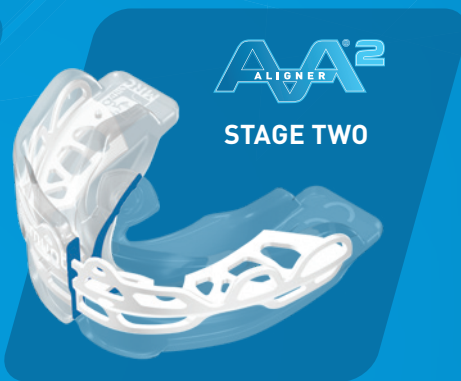

AA<sup>2</sup>  
ALIGNER

STAGE TWO



AA<sup>3</sup>  
ALIGNER


STAGE THREE



For more information please scan the QR code  
or visit [myoresearch.com/adult-aligner](http://myoresearch.com/adult-aligner)



## Stage 1 – Habit correction

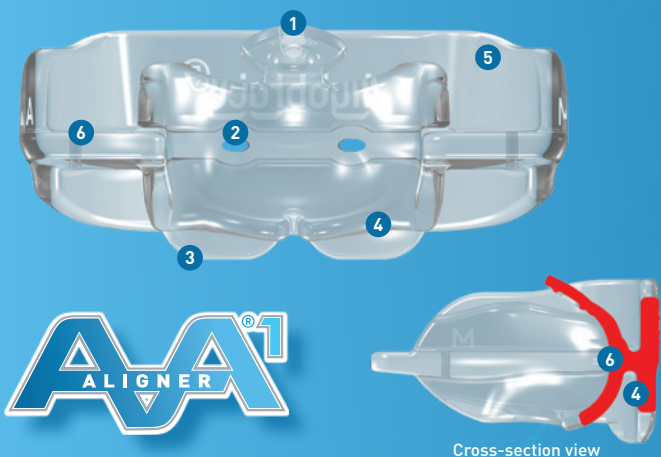


The *Myobrace*® AA1 is the first stage in the *Adult Aligner* appliance group and specifically focuses on initial habit correction. It works on establishing nasal breathing and initial correction of poor oral habits, including training the tongue to rest in the correct position.

The AA1 is soft and flexible to easily adapt to crowded teeth, making it an ideal starting appliance. It uses light forces to align the teeth and bite in harmony with the oral muscles.

- 1 **Flexible tongue tag** guides the tongue upwards and forwards to the correct position.
- 2 **Breathing holes** encourage and establish nasal breathing.
- 3 **Lip bumper** corrects dysfunctional swallowing.
- 4 **Tongue elevator** lifts the tongue into the correct position.
- 5 **Flexible material** adapts to any arch form and improves comfort.
- 6 **Tooth channels** align the teeth.

## Stage 2 – Arch development

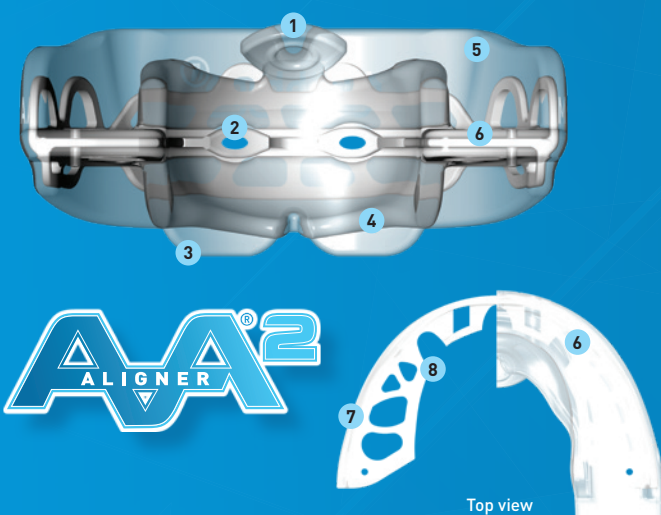


The *Myobrace*® AA2 is the second stage in the *Adult Aligner* appliance group and specifically focuses on arch development (to make more room for the tongue while simultaneously training it to rest in the correct position), and continuing habit correction to improve other poor myofunctional habits.

The AA2 has a uniquely designed rigid inner core, the *Dynamicore*™, that can adapt to narrow arches, creating force to develop the jaws and align the teeth. The soft and flexible outer layer maintains comfort.

- 1 **Flexible tongue tag** guides the tongue upwards and forwards to the correct position.
- 2 **Breathing holes** encourage and establish nasal breathing.
- 3 **Lip bumper** corrects dysfunctional swallowing.
- 4 **Tongue elevator** lifts the tongue into the correct position.
- 5 **Soft outer layer** optimises and maintains comfort.
- 6 **Tooth channels** align the teeth.
- 7 **Unique *Dynamicore*™** promotes extra arch development in adults.
- 8 **Cutouts** for extra adaptability to the natural arch form.

## Stage 3 – Final alignment and retention

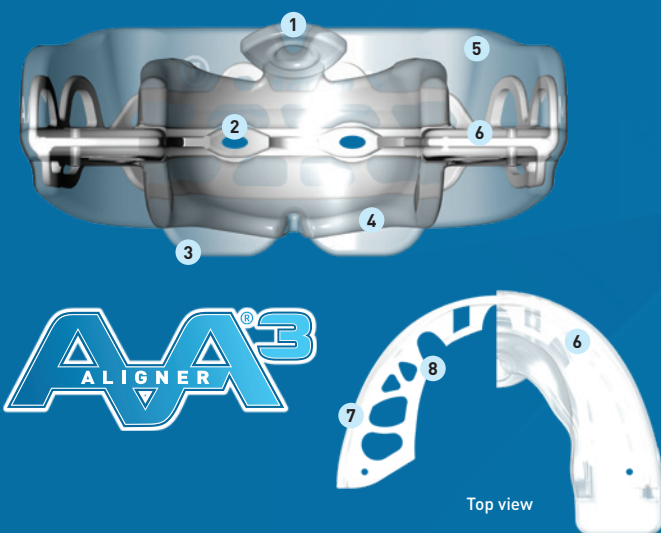

The *Myobrace*® AA3 is the third and final stage in the *Adult Aligner* appliance group and specifically focuses on final dental alignment and retention of the arch form.

The AA3 is the hardest appliance in the group, with a uniquely designed rigid inner core and a firm but comfortable outer layer, that enables it to apply additional force. The AA3 features ensure that breathing and myofunctional habits are retained and improved further (if required). The *Myoretain*® can be used after treatment for further retention.

- 1 **Flexible tongue tag** guides the tongue upwards and forwards to the correct position.
- 2 **Breathing holes** encourage and establish nasal breathing.
- 3 **Lip bumper** corrects dysfunctional swallowing.
- 4 **Tongue elevator** lifts the tongue into the correct position.
- 5 **Firm outer layer** increases rigidity but maintains comfort.
- 6 **Tooth channels** align the teeth.
- 7 **Unique *Dynamicore*™** provides further rigidity and promotes retention.
- 8 **Cutouts** for extra adaptability to the natural arch form.