

Specifically designed for the
Mouth Breather and Snorer



TMJ•MBV™
MOUTH BREATHER VERSION

www.myoresearch.com

WORLDWIDE PATENTS, FOR MORE INFORMATION VISIT MYORESEARCH.COM

BIGGER SIZE FOR BETTER RETENTION

The *TMJ-MBV* is specifically designed for mouth breathers who suffer from TMJ disorder symptoms. The higher vertical sides of the *TMJ-MBV* provide better comfort and retention, particularly when the mouth is open.

TMJ-MBV FEATURES

- 1 **Four large breathing holes** allow for easy breathing with the mouth open.
- 2 **Tongue guard** controls the tongue position.
- 3 **Higher sides** for optimum retention.
- 4 **Air spring base** for variable bite opening and gentle TM Joint decompression.



Cleaning your *TMJ-MBV*: Run your *TMJ-MBV* under warm water every time you remove it from your mouth and allow it to dry. We recommend using *Myoclean*™ tablets for all your oral appliances twice a week. Ask your practitioner about *Myoclean*™.

CAUTION: USE UNDER THE SUPERVISION OF AN ORTHODONTIST OR DENTIST.

Lifetime use: 6 months based on everyday wear. Replace after 12 months regardless of wear time. At the first signs of appliance cracking, damage or discolouration, speak to your health practitioner.



TMJ-MBV™ by Myofunctional Research Co.
Myofunctional Research Co. Australia
44 Siganto Drive, Helensvale QLD 4212 Australia
Tel: +61 7 5573 5999

USA Representative:

Myofunctional Research Co. USA
9267 Charles Smith Avenue, Rancho Cucamonga CA 91730 USA
PO Box 2781, Rancho Cucamonga CA 91730 USA
Tel: +1 866 550 4696



Europe Representative:

Myofunctional Research Europe B.V.
Gompensstraat 21c 5145 RM Waalwijk The Netherlands
P.O. Box 718, 5140 AS Waalwijk The Netherlands
Tel: +31 416 651 696



MYOFUNCTIONAL RESEARCH CO.

INNOVATIVE DENTAL APPLIANCE TECHNOLOGY AND EDUCATION



SINGLE PATIENT
MULTIPLE USE



KEEP OUT
OF DIRECT
SUNLIGHT



NON
STERILE
CLEAN
BEFORE USE